

Menu button continued





Barrel Change Sequence - Sub menu

The barrel change sequence is the last item accessed from the menu button. Used to change the barrel size back and forth between the 70ml (Dosing) and 14ml (injection) barrel types. When you select the tick and press the menu button on this item you enter the barrel change sub menu as below. Pressing and holding the Menu Button at any time returns you to the main screen.

Enter the barrel change sub-menu as described above and follow the sequence as displayed as follows:



Remove the barrel

The arrow flashes on the screen. Remove the barrel collar and remove the barrel. Press the menu button for the next step.



Remove the piston

The arrows flash on the screen. Remove the piston by holding down the trigger of the gun and gently pulling it out.

Press the menu button for the next step.



Load the piston for the smaller barrel

The arrows flash on the screen. Load the piston by holding down the trigger of the gun and gently pressing it in. The piston stops in the "Home" position and the screen automatically advances to the next step.



Select the barrel size you are fitting

Use the arrow keys to select the size barrel you are installing. Then press the menu button to proceed to the next item.

Ensure you have the correct barrel and piston size for the setting. If the setting is not correct to your installed barrel and piston, damage may occur.



Barrel change sub menu continued



Fit the optional barrel

Fit the new barrel to the gun with the collar.

Press the menu button to complete the sequence



Barrel change complete

The screen displays a gun icon with a tick to let you know the process is complete. The gun saves your changes and returns automatically to the main screen. Note: The fidelity of the dose amounts changes when using the 14ml barrel to .5 ml increments.

Barrel size, dose amount and delivery specification

70 ml Barrel:

5 to 99 ml; Dose volume increments of 1 ml; 71 ml to 99 ml delivered in 2 sequential shots.

14 ml Barrel:

1 to 28 ml; Dose volume increments of .5 ml; 14.5 ml to 28 ml delivered in 2 sequential shots.

Sequential shots:

These are delivered automatically and immediately after each other, with just the one trigger click.

Notes	