


THE DIGITAL KEYPAD

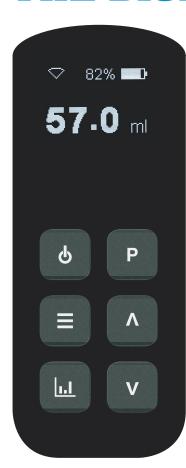

Main screen

The top left shows a Wi-Fi icon with bars when it is active. The top right shows the battery state and in the main area the current dose.



Power button

Click once to turn on, press and hold momentarily to turn off.



Prime button

Push this button and pull the trigger to fill the barrel. Point the Dosing gun up to expel any air in the barrel. This gives manual control of the piston. Click again to exit this mode.



Menu button

Use this button to adjust the delivery and refill speed. Also use it to turn on sound alert, repeat dose delay, multi dose (max 10ml dose) and barrel change. The Menu button also functions as the Select button.



Statistics button

Push to see the amount of product administered and number of doses



Up and Down buttons



Push these buttons to adjust the dose volume.

Batteries

Take note of the polarity of the batteries. Remove the 2 batteries from the bottom of the Dosing Gun and insert into charger.

Maintenance:

It is imperative that you clean the barrel after every session. Remove the barrel, rinse the barrel with clean, warm water and dry it thoroughly. Remove the inlet and exit valves and clean thoroughly. Lubricate the piston O-ring with petroleum jelly or food/cooking oil.

FOR SERVICE OR REPAIRS CONTACT:

New Zealand: Australia: 0800 837 274 1800 650 682 United States: Rest of world:

800 315 0433 +64 3 433 0077

