

EZE BLOODLESS CASTRATION INSTRUCTIONS

	Place under arm pit		Load clip with notch up and forward
	Slide over pull rod. Also pull the pull rod back so it is out of the way for easier installation		Place the clip in the slot so it is completely in and flush
	Hold the band with the thumb on top and the index finger on the end of the spreader bar to keep from slipping off		Continue holding while stretching band with other hand
	Stretch to other spreader bar and attach. When stretching try to stretch upper portion as tight as possible leaving lower portion with as much slack as possible		Hold band with both thumbs on top and both index fingers in the end of spreader bars
	Start stretching lower portion of band with remaining fingers		Continue until band is in groove of pull back rod
	Ensure band isn't twisted when attaching to pull back rod		Hold castrator by the side grip handle. Side grip may be removed and placed on other side
	With other hand, pull the rod back until the band slips through the hole where you placed the clip		Make sure the castrator is facing upright since the clip may slip out of the slot
			Stop once the end of the rod lines up with the back side of the slot

TE PARI PRODUCTS

NEW ZEALAND:

PO BOX 25, 9 ENDEAVOUR CRESCENT, OAMARU 9400.
Ph: +64 3 433 0077 Em: sales.nz@tepari.com
FREEPHONE 0800 837 274

AUSTRALIA:

44 RICKY WAY, EPPING 3076 VICTORIA.
Fx: 1800 669 205 Em: sales.au@tepari.com
FREEPHONE 1800 650 682

USA:

14250 JUDICIAL ROAD, BURNSVILLE, MN 55306, USA.
Em: sales.usa@tepari.com
FREEPHONE 800 315 0433

	<p>Castrator is fully loaded</p>		<p>Firmly grip the scrotum and squeeze it a couple of times to work some of the fatty tissue downward</p>
	<p>Continue hold the scrotum. Slip the scrotum through the band. Placing the band just above the testicles.</p>		<p>DO NOT PLACE BAND NEXT TO THE BELLY – Just below it.</p>
	<p>Release the band from the castrator Continue holding the scrotum – Pull (jerk) the castrator back with the other hand. Don't let the scrotum move while performing this</p>		<p>Let the base of the castrator rest next to the scrotum and let the scrotum hang naturally. No need to pull back on the castrator</p>
	<p>Place one hand on the side grip and one on the pull back rod. Pull back the rod as far as possible.</p>		<p>Take care not to hit the blade handle. Continue tightening with pistol grip handle</p>
	<p>There is a small stop weld on the rod which will stop the rod from tightening any further.</p>		<p>Use the crimp handle to crimp the retaining clip. Crimp until the handle stops</p>
	<p>Use the blade handle to cut the band. Follow through completely</p>		<p>Top view of blade cutting band</p>
	<p>Now jerk back on the castrator and pull the retaining clip from the slot</p>		<p>Castrator completely removed from scrotum</p>
	<p>Inspect band to ensure no twists and not to close to belly</p>		<p>Inspect clip to ensure it is on correctly</p>

TE PARI PRODUCTS

NEW ZEALAND:

PO BOX 25, 9 ENDEAVOUR CRESCENT, OAMARU 9400.
Ph: +64 3 433 0077 Em: sales.nz@tepari.com
FREEPHONE 0800 837 274

AUSTRALIA:

44 RICKY WAY, EPPING 3076 VICTORIA.
Fx: 1800 669 205 Em: sales.au@tepari.com
FREEPHONE 1800 650 682

USA:

14250 JUDICIAL ROAD, BURNSVILLE, MN 55306, USA.
Em: sales.usa@tepari.com
FREEPHONE 800 315 0433