

Connecting the Gallagher TSi2 to Te Pari Dosing Gun APP


(NOTE: For Android only via Bluetooth)

Firstly, to connect the TSi to the Android device you will need a Bluetooth dongle (G03203) which will plug into an available serial Port on the TSi indicator.

Once BT Dongle plugged into TSi go to Options

- > Connect Equipment
- > Select the Serial Port the BT dongle is Plugged into ie PORT 1 (not Bluetooth Port)
- > Change
- > Data Logger
- > Show Settings
- > Baud Rate = 9600 (Note Tsi is 19200 by default and BT Dongle is 9600 by default)
- > Handshaking = None (Tsi is Hardware by default)
- > Options – Send On Weight Lock
- > Select OK
- > Back (To Yard Screen)

See correct settings below:

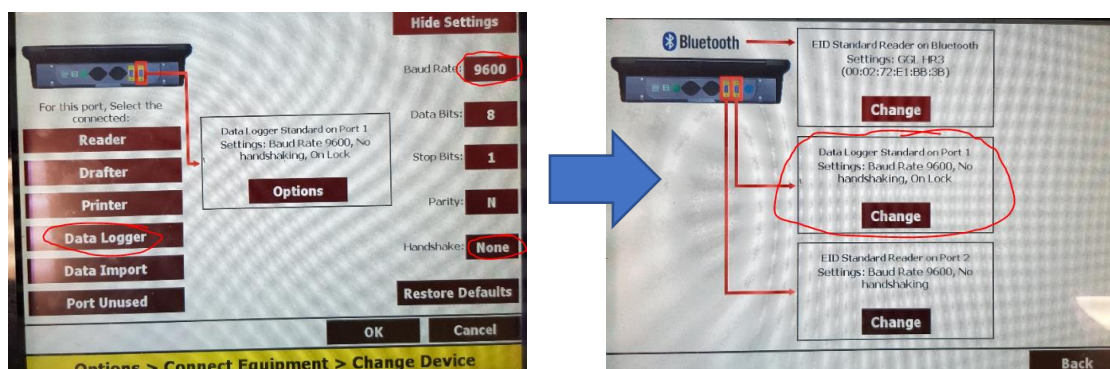

TE PARI PRODUCTS

NEW ZEALAND:
 PO BOX 25, 9 ENDEAVOUR CRESCENT, OAMARU 9400.
 Ph: +64 3 433 0077 Em: sales.nz@tepari.com
 FREEPHONE 0800 837 274

AUSTRALIA:
 44 RICKY WAY, EPPING 3076 VICTORIA.
 Fx: 1800 669 205 Em: sales.au@tepari.com
 FREEPHONE 1800 650 682

USA:
 14250 JUDICIAL ROAD, BURNSVILLE, MN 55306, USA.
 Em: sales.usa@tepari.com
 FREEPHONE 800 315 0433


On the Android device Go to Bluetooth Settings

> Search for Available devices

> Select ' RS232 Dongle ' > Enter Pin ' 0000 ' (ie 4 number zeroes)

> Download and Install Te Pari Dosing Gun App from App Store.

Open App > Go to Settings > Change Scale Brand to 'Gallagher '

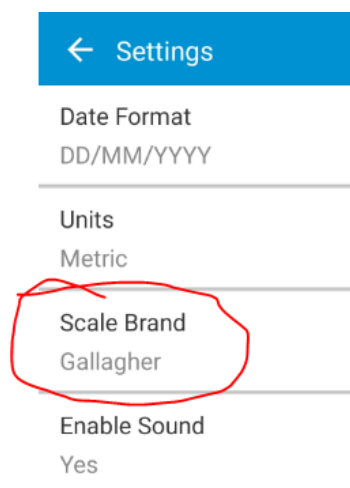

TE PARI PRODUCTS

NEW ZEALAND:

PO BOX 25, 9 ENDEAVOUR CRESCENT, OAMARU 9400.
Ph: +64 3 433 0077 Em: sales.nz@tepari.com
FREEPHONE 0800 837 274

AUSTRALIA:


44 RICKY WAY, EPPING 3076 VICTORIA.
Fx: 1800 669 205 Em: sales.au@tepari.com
FREEPHONE 1800 650 682

USA:

14250 JUDICIAL ROAD, BURNSVILLE, MN 55306, USA.
Em: sales.usa@tepari.com
FREEPHONE 800 315 0433

In the App Go to New Session

- > Enter Required Fields * (i.e. Session name and Dosage Rate mls & KGs)
- > Select Save & Start > Select Connect Weigh Scale > Connect > Locate RS232 Dongle from the list and Select it, NOTE at this point the Blue light should now be activated on the BT Dongle.

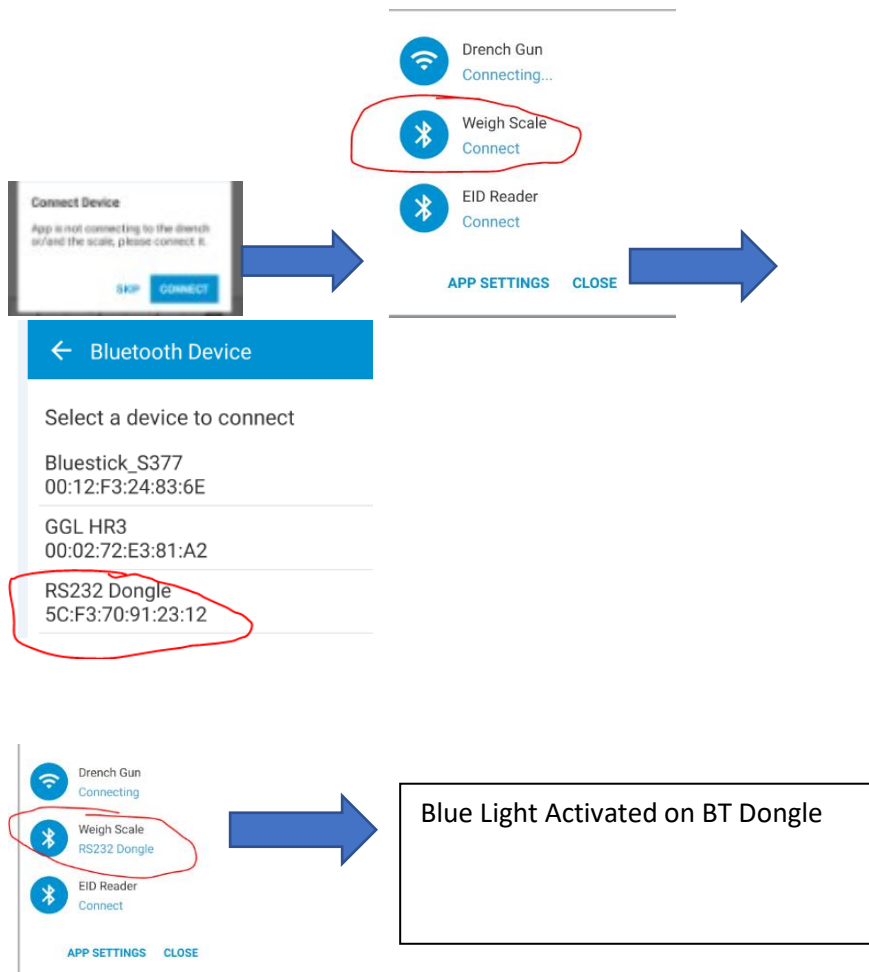

TE PARI PRODUCTS

NEW ZEALAND:
 PO BOX 25, 9 ENDEAVOUR CRESCENT, OAMARU 9400.
 Ph: +64 3 433 0077 Em: sales.nz@tepari.com
 FREEPHONE 0800 837 274


AUSTRALIA:
 44 RICKY WAY, EPPING 3076 VICTORIA.
 Fx: 1800 669 205 Em: sales.au@tepari.com
 FREEPHONE 1800 650 682

USA:
 14250 JUDICIAL ROAD, BURNSVILLE, MN 55306, USA.
 Em: sales.usa@tepari.com
 FREEPHONE 800 315 0433

At this point you can connect to the Drench Gun which uses Wi-Fi, please follow manufacturer's instructions for this.

On the TSi do the following;

- > Loadbars plugged in
- > Options > Create a New Session > Full Animal ID
- > Scan a Tag (assuming EID reader is connected to TSi)
- > Lock in a Weight
- > On the APP the Locked weight will appear screen and Dose to apply.

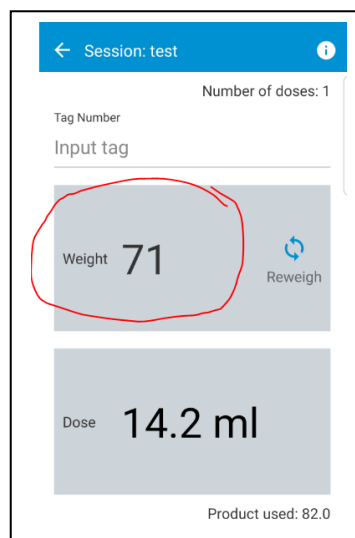

on the

TE PARI PRODUCTS

NEW ZEALAND:
PO BOX 25, 9 ENDEAVOUR CRESCENT, OAMARU 9400.
Ph: +64 3 433 0077 Em: sales.nz@tepari.com
FREEPHONE 0800 837 274

AUSTRALIA:
44 RICKY WAY, EPPING 3076 VICTORIA.
Fx: 1800 669 205 Em: sales.au@tepari.com
FREEPHONE 1800 650 682

USA:
14250 JUDICIAL ROAD, BURNSVILLE, MN 55306, USA.
Em: sales.usa@tepari.com
FREEPHONE 800 315 0433